

# VIVA! Medipro

[www.summitbaby.com](http://www.summitbaby.com)

## **Material**

Polyurethane Transfer Laminated Polyester Knit engineered for Hospital use in Europe and USA. Content: 52% Polyurethane surface laminate, 48% Polyester backing.

## **Recommended Cleaning Instructions:**

### **Superficial dirt**

Use a disposable wipe and warm solution of neutral detergent. Do not use abrasive cleaners

### **Disinfect**

Disinfect in situ using 0.1% sodium hypochlorite solution (1,000 ppm chlorine). Alternatively use proprietary disinfectants according to the manufacturers instructions.

### **Launder**

Launder in process validated washing machine operating at 65C° (149F°) for not less than 10 minutes or at 70C° (158F°) for not less than 3 minutes (NHS Guidelines). Do not mangle. Spin and tumble-dry at not more than 130C° (266F°). Some surface wrinkling may occur which has no adverse effect on the properties of the fabric. Do not iron.

### **Autoclaving**

Fabrics can be autoclaved at 130C° (266F°)

### **Storage**

Thoroughly dry fabric before re-use or storage. Do not fold away wet or store in damp conditions.

### **Vinyl Under Pillow**

Surface clean with mild soap and warm water. Wipe or rinse soap away. Disinfect by wiping with a bleach (sodium hypochlorite) solution; 10% bleach, 90% water. Rub gently. Rinse with a water to remove bleach concentration. Alternatively use proprietary disinfectants according to the manufacturers instructions.

UNDER PENALTY OF LAW  
THIS TAG NOT TO BE REMOVED  
EXCEPT BY THE CONSUMER

ALL NEW MATERIAL  
CONSISTING OF

### **VINYL UNDERPILLOW:**

MADE IN CHINA

COMPLIES ASTM STANDARD

F963-96A FOR PVC

### **FABRIC COVER:**

POLYURETHANE

TRANSFER LAMINATE

MADE IN EUROPEAN UNION

MANUFACTURER/ PRODUCER:

**SUMMIT BABY INC.**

**PO BOX 1214**

**SUMMIT, NJ 07901**

**USA**

[www.summitbaby.com](http://www.summitbaby.com)

[info@summitbaby.com](mailto:info@summitbaby.com)

**ID# 0400073988**

**PATENT PENDING**

### **CAUTION**

HOLD BABY IN ARMS

AT ALL TIMES

DO NOT STAND OR WALK

WHILE USING THE PILLOW